

Planmeca Romexis® implant installer

Quick Guide

Romexis® implant installer

Download implant manufacturer specific implant installation packages from <https://www.planmeca.com/software/software-modules/planmeca-romexis-3d-implantology/implant-library/>
The package includes both Windows and macOS installers.

Note:

- Romexis 5.0 or newer is required
- Romexis 3D Implant module is required
- Installation needs to be done on the Romexis server computer
- Implant library is not available for the free Romexis Viewer

Instructions

Go to <https://www.planmeca.com/software/software-modules/planmeca-romexis-3d-implantology/implant-library/>.

- 1** Select the implant manufacturer and click **DOWNLOAD LIBRARY**.

Adin

Library contains:
- Implants, Surgical kit

Update info:
11.1.2022 Surgical kit: New manufacturer

DOWNLOAD LIBRARY >

- 2** Browse to the .zip file.

Windows:

- a Right-click the .zip file
- b Choose "Extract All...".
- c Open the folder that was extracted and start installation by double-clicking **ImplantLibraryInstaller.exe**

MacOs:

- a Extract by double-clicking the .zip file.
- b Open the folder that was extracted.
- c Double-click **ImplantLibraryInstaller_OSX**

3 Select installation

Basic installation:



Select basic installation if Romexis has been installed to default folder. In the first window:

- a Click **Continue**.
- b Click **Install**.

Custom installation:

If Romexis has been installed to a custom location or database username and password has been changed select the *Custom installation* option.

- a Browse to the location of Romexis server.
- b Specify database username and password.

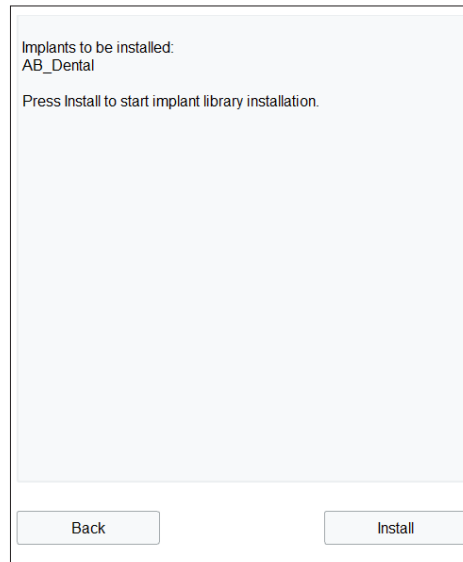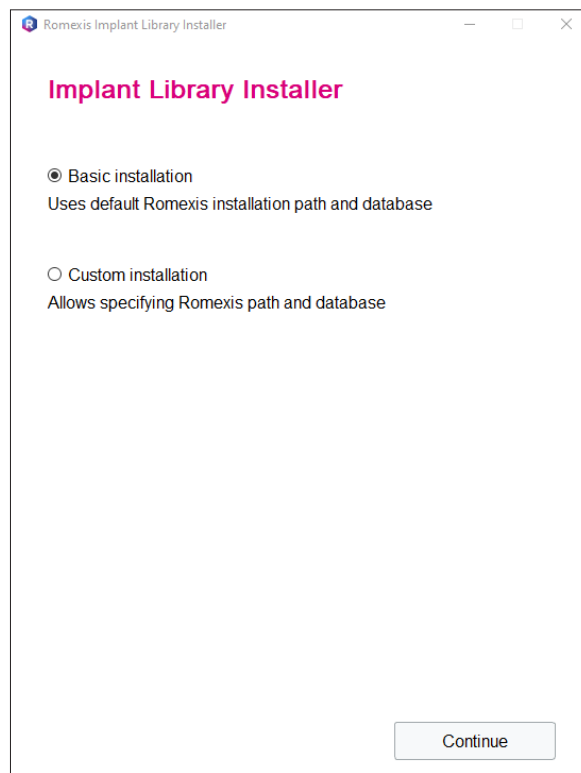

4 Restart Romexis

The implant library is now updated.

For detailed instructions, please see the user's manual. Rights for changes reserved.